2020/04/28/SG

Notes for users on the handling of self made face masks ("community masks")

The best protection against a potential virus transmission is still to consistently distance oneself from other, potentially virus-bearing persons. Nevertheless, the physical barrier of correctly wearing a community mask can provide some protection against larger droplets and oral/nasal mucous membrane contact with contaminated hands.

Persons who wear a community mask should therefore be sure to observe the following rules:

- The current hygiene regulations, in particular the current recommendations of the Robert Koch Institute (RKI, www.rki.de) and the Federal Centre for Health Education (BZgA, www.infektionsschutz.de) must still be observed.
- Even with a mask, the safety distance of at least 1.50 m from other people recommended by the WHO should be maintained.
- When putting on a mask, care must be taken to ensure that the inside is not contaminated. Hands should be washed thoroughly with soap before putting on the mask.
- The mask must be positioned correctly over the mouth, nose and cheeks and must fit as tightly as possible around the edges to minimise air ingress at the sides.
- The first time you use the mask, check that it allows enough air to pass through to minimise the risk of obstructing normal breathing.
- A soaked mask should be removed immediately and replaced if necessary.
- The outside of the used mask is potentially infectious. To prevent contamination of the hands, do not touch them.
- When the mask is removed, hands should be washed thoroughly (with soap for at least 20-30 seconds) in accordance with general hygiene rules.
- After removal, the mask should be stored in an airtight bag or similar, or washed immediately. It should only be stored for as short a time as possible to avoid the formation of mould in particular.
- After a single use, masks should ideally be washed at 95 degrees, but at least at 60 degrees, and then dried completely. Observe any manufacturer's instructions regarding the maximum number of cycles after which the strength and functionality is still given.
- If available, all manufacturer's instructions should be observed.

Translated with www.DeepL.com/Translator (free version)

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Putting on and taking off a face mask

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Putting on your face mask

- define in- and outside of the mask, and don't change them
- the mask should have an integrated nose clip

- 1./ 2. wash and dry your hands before putting on your face mask
3. proper alignment: the nose clip has to be at the top (for medical masks, the white side is inside)
4. unfold the mask

Modelling of your mask

- 5a. If your masks has a twin strap design, pull them over your ears. Press the nose clip to your nose and tear the mask down to your chin.
- 5b. If your masks has ribbons, fix at first the nose clip, then tie the ribbons, that the mask stays in place and tear the mask under your chin.
6. correct fit: Mask must fit tight and cover mouth and nose completely.

Taking off your face mask

Don't touch your mucosa membranes in your eyes, nose or mouth while taking off the face mask.

Don't touch the front part of the mask, take it off by picking it at the ribbons or the straps.

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Cleaning of self-sewn masks

Washing

- as often as possible with 60°
- let them dry outside

Oven

- at 80° for 10 minutes

Microwave

- Place mask with outside over water bowl
- high wattage, several minutes
(disinfection by steaming)

Electric iron

- if it has to be done quickly, just iron it over
- better: washing and ironing

(From: <https://www.zdf.de/nachrichten/briefing/coronavirus-maskenpflicht-zdfheute-update-100.html>, 27.4.2020)